


EP David Biosense Webster

CLEANING GUIDE

1. Switch **OFF** the EP David.
3. Connect one end of the supplied silicone tube to the indicated connector (blue circle) and put the other one in an external container.

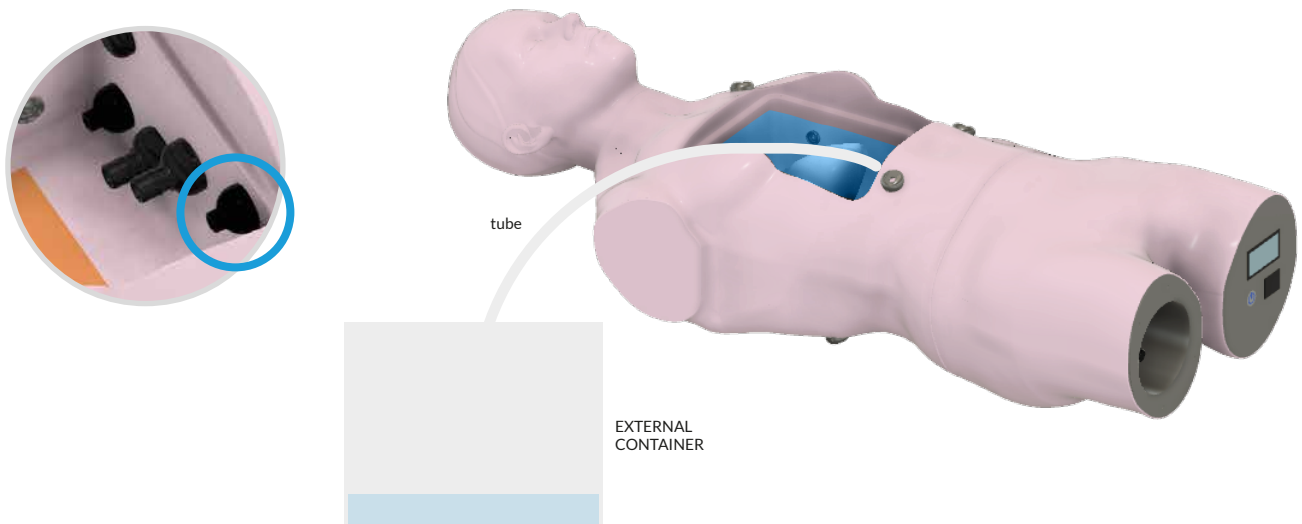

4. Switch back **ON** the EP David to start the water flow.
5. Once the water reached the minimum level, the flow from the tube will stop.
Please consider that the pump won't empty the EP David completely .
5. Switch off the EP David.
6. Disconnect all the cables (6 patches, 1 indifferent electrode, power cable) and the silicone tube.
Secure each cable with the supplied straps.
7. Discard the remaining water.
8. Clean the system with paper towels.
9. It's time to pack everything up! Follow the **Packing Guide** to store all the components safely.