


EP Tank Biosense Webster

CLEANING GUIDE


1. Switch off the EP Tank
2. Disconnect all the cables (6 patches, 1 indifferent electrode, power cable)
3. Remove the meat, the mini table and the catheters support
4. Empty the EP Tank using the provided external pump (if not provided skip to **STEP 4.2**)
 - connect the supplied tube to the pump outlet
 - place the pump in the water and the other side of the tube in an external container
 - connect the pump to the AC outlet (**BE CAREFUL!** The flow will start as soon as the plug is connected)

Please consider that the pump won't empty completely the EP Tank.

Once you reached the pump level you can switch off the pump and discard the remaining water.

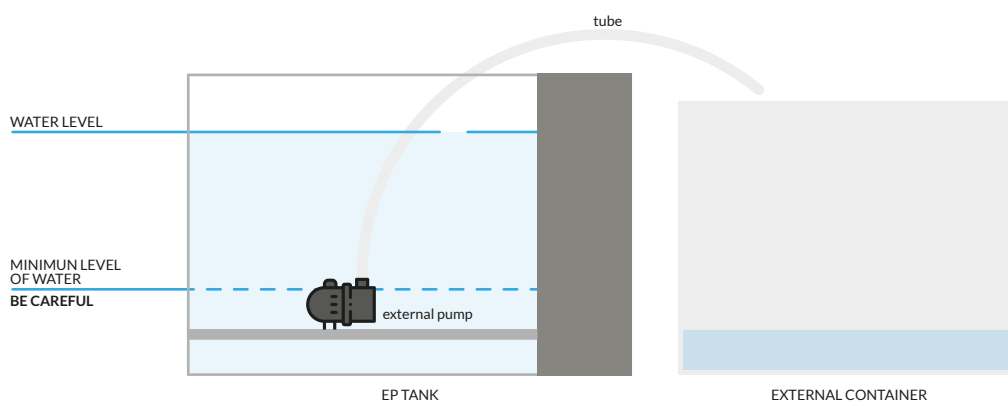

4.2 Connect the silicon tube to the pump outlet

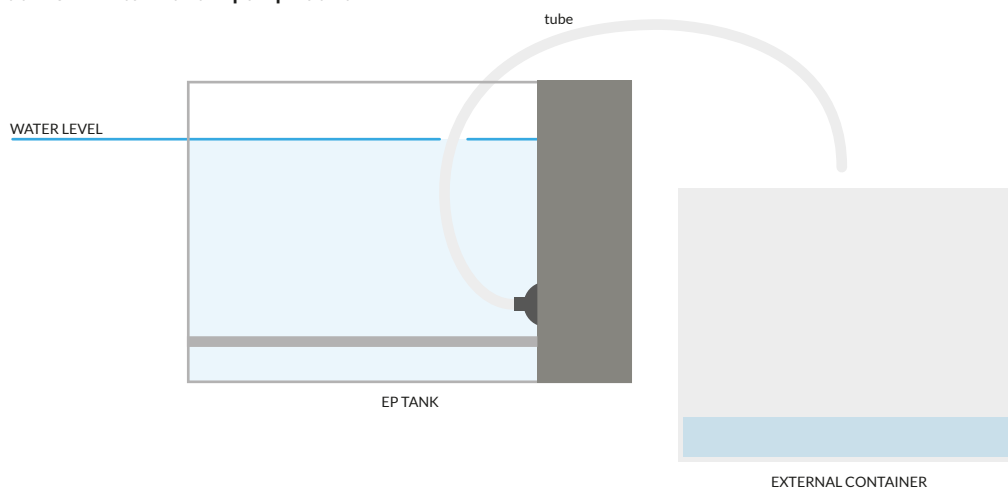

TIPS.

To empty the EP Tank faster place the external container on a lower level than the EP Tank. Once the flow from the pump is started, remove the tube from the connector and keep it in the EP Tank to remove the water. As soon as the flow stops you can discard the remaining water.

5. Rinse the EP Tank with clean water and get it dry with the provided towel (don't use ethyl alcohol, just use soapy water)
6. It's time to pack up everything! Follow the **Packing Guide** to pack all the components safely.